

Operation Reminders

To ensure the smooth operation of the system and mitigate potential risks, users should take note of the following reminders when operating the system:

1. Incorrect Data Input (U1)

Issue: Inputting incorrect data may lead to difficulties in locating the correct data or images.

Reminder: When entering data, carefully heed system warnings and ensure the accuracy of patient IDs and relevant numbers.

2. Measurement Deviation (U2)

Issue: Deviations in imaging measurements may cause doctors to inaccurately assess lesions.

Reminder: Use the calibration function to calibrate the currently displayed image and ensure precise image dimensions.

3. Incorrect System Feedback (U3)

Issue: Incorrect system feedback may inconvenience patients and doctors.

Reminder: Use the original manufacturer-provided database system for reliable system performance.

4. System hang-up (U4)

Issue: System hang-ups may disrupt data or image processing, causing inconvenience to patients and doctors.

Reminder: Verify that the software and hardware configurations align with the specifications outlined in the manual. In the event of a hang-up, follow the system prompts to restart and redo the operation.

5. Unauthorized Operations (U5)

Issue: Unauthorized operations may compromise system information integrity, causing inconvenience to patients and doctors.

Reminder: Users must receive proper training and obtain authorization before using the system. Unauthorized access is strictly prohibited.

6. Database Damage or Data Loss (U6)

Issue: Database damage or data loss may impede access to images and data, wasting time and inconveniencing patients and doctors.

Reminder: Use the original manufacturer-provided database system and regularly backup database files.

7. Insufficient Memory Capacity (U7)

Issue: Insufficient memory capacity may result in system slowdowns or interruptions.

Reminder: Ensure adequate memory allocation according to the requirements outlined in the manual.

8. Lost Token (U8)

Issue: Losing the token will render the system unusable.

Reminder: Safeguard the token securely. In case of loss, contact the original vendor for assistance in obtaining a replacement.

9. Power Supply Issues (S1)

Issue: Power supply interruptions or instability may disrupt operations and data transmission.

Reminder: Consider installing an uninterrupted power supply (UPS) to mitigate interruptions. In the event of a power outage, follow the system prompts to resume operation.